## **BS-600**

## **Chemistry Analyzer**

#### **Technical Specifications**

System function

Fully automated, discrete, random access,

STAT, urine and homogeneous immunoassays;STAT sample priority Throughput: 600 photometric tests/hour, up to 770 tests/hour for

Measuring principles: Absorbance Photometry, Turbidimetry Methodology: End-point, Fixed-time, Kinetic, optional ISE Single/Dual/Triple/Quadruple reagent chemistries,

Programming: User defined profiles and calculation

Sample Handling

Monochromatic/Bichromatic

Sample tray: 90 positions for primary or secondary tubes and

sample cups

Sample volume: 1.5~45 μl, step by 0.1μl

Sample probe: Liquid level detection, clot detection and collision

Probe cleaning: Interior and exterior automatic probe washing carry-over < 0.05%

Automatic sample dilution, Pre-dilution and post-dilution

Internal bar code reader (optional)

Sample/Reagent barcode reading – applicable to various bar code systems including Codabar, ITF (Interleaved Two of Five), code 128, code 39,

UPC/EAN, Code93; Bi-directional LIS Interface transmission

ISE Module (optional)

Optional selection of K+, Na+, Cl Throughput: Up to 255 tests per hour

Reagent Handling

Reagent tray: 80 positions in refrigerated compartment (2~10°C) Reagent volume:  $10\sim200\mu l$ , step by  $0.5\mu L$ 

Reagent probe: Liquid level detection, collision protection and

inventory check, reagent bubble detection

 $\label{probe} Probe\ cleaning: Interior\ and\ exterior\ automatic\ probe\ washing$ 

Reaction System

Reaction rotor: Rotating tray, 124 cuvettes with automatic

washin

Cuvette: Optical length 5mm Reaction volume: 100~300µl Operating temperature: 37°C

Mixing system: 2 independent mixers

Optical System

Light Source: Halogen-tungsten lamp

Photometer: Reversed optics, grating photometry

Wavelength: 340nm, 380nm, 412nm, 450nm, 505nm, 546nm,

570nm, 605nm, 660nm, 700nm, 740nm, 800nm Absorbance range: 0~3.3Abs (10mm conversion)

Resolution: 0.0001Abs

Control and Calibration

Calibration mode: Linear (one-point, two-point and

multi-point),Logit

-Log 4P, Logit-Log 5P, Spline, exponential, Polynomial, Parabola Control rules: Westgard multi-rule, Levy-Jennings, Cumulative sum check, Twin plot

Operation Unit

Operation system: Window XP Professional or Windows 7

Professional or Windows 8,

Interface: RS-232, Network Port, USB/ parallel port

**Working Condition** 

Power Supply: 220V, 50/60Hz; ≤1700VA

Temperature: 15~30°C

Humidity: 35~85%

Water consumption: ≤28L/H, De-ionized water Dimension: 1190mm(W)\*1155mm(H)\*720mm(D)

Weight: 300 Kg

Mindray Building, Keji 12th Road South, High-tech Industrial Park, Nanshan, Shenzhen 518057, P.R. China Tel: +86 755 8188 8998 Fax: +86 755 26582680 E-mail: intl-market@mindray.com www.mindray.com

Mindray is listed on the NYSE under the symbol "MR"

mindray are registered trademarks or trademarks owned by Shenzhen Mindray Bio-medical Electronics Co., LTD.

© 2015 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved. Specifications subject to changes without prior notice.

P/N: ENG-BS-600-210285x8-20150514





**BS-600** Chemistry Analyzer





## **BS-600**

**Chemistry Analyzer** 



## **Easy maintenance**

- All containers & maintenance kits are located in the front of the analyzer
- Easily accessible for part replacement, routine maintenance or troubleshooting

### **Reagent bubble detection**

 Able to detect bubbles in reagent bottles, as well as detect real liquid level before reagent aspiration







## Whole blood HbA1C testing\*

- No need pretreatment of the sample
- Proven precision and specificity
- No interference from hemoglobin variants
- Traceable to IFCC/NGSP reference methods

#### **Cost-effective**

- $\bullet~$  Sampling precision up to 1.5  $\mu L$
- Perfect match between instrumentation and dedicated reagents
- 100µL minimum reaction volume ensures more effective cost per test

## **Lower carry over**

- High pressure washing for interior and water fall washing for exterior
- Carry over < 0.05%

\* in development

# **BS-600**

## **Chemistry Analyzer**

#### Advanced software

### **User-friendly interface**

- Touch screen
- Share the same platform with BS-2000 series and BS-800 series
- Real-time status monitoring between analytical unit and carousels





## Convenient R/S continuous loading and offloading


 Remaining time counter makes it easy for operator to follow the instructions

## Intelligent reagent management

- · Real-time indicating the test number, inventory and expiry date
- Improve work efficiency

#### **Reflex function**

- Pre-defined reflexive assays will be performed automatically when preset criteria is met
- Each assay may involve multiple reflexive criteria
- Each criteria may initiate up to a maximum of 20 relevant assays

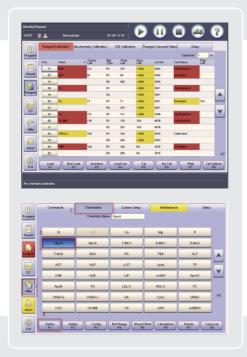




#### **Test Summary**

- Test summary report calibration, QC, sample, validation test and rerun tests can be generated
- Facilitate computation of total test costs
- Error Log Export function -facilitate error report to engineers
- Results Archive can be transferred to engineers for evaluation

## Mindray solution for clinical chemistry

After more than 10 years of research and development on reagents, Mindray can now provide 61 parameters of dedicated reagents, covering hepatic, renal, cardiac, lipids, diabetes, pancreatitis, inorganic ions and immunoassays, etc., together with original calibrators with metrological traceability as well as controls for BS-600 chemistry analyzer.

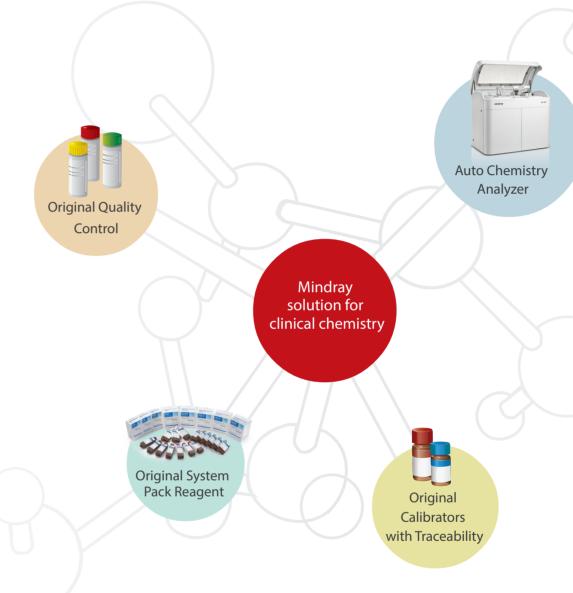

## Standard reference system

- Adopt JCTLM reference system
- IFCC primary method for enzyme, ID/MS method for substrate
- NIST, IRMM reference materials

JCTLM, Joint Committee On Traceability In Laboratory Medicine

NIST, National Institute of Standards and Technology, USA


IRMM, Institute for Reference Materials and Measurements, EU

IFCC, International Federation of Clinical Chemistry and Laboratory Medicine

#### **Complete traceability process**

Complete calibration hierarchy and traceability chain based on ISO standard (EN/ISO17511) from reference system to routine measurement system

#### Traceability chain of Mindray measurement system (Glu)

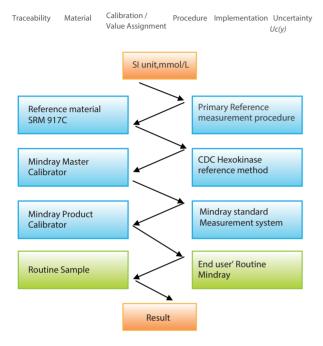


# External quality assurance for reference measurement

Mindray participates RELA (External quality control for reference laboratory) and CAP (College of American Pathologists external quality control)

#### **EQA for Mindray Reference laboratory—— Rela**

Mindray reference laboratory has passed Rela for 6 continuous years

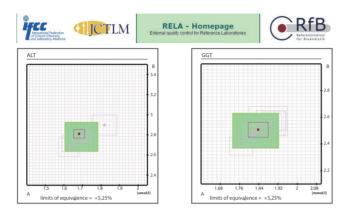


More Rela results please refer to: www.dgkl-rfb.de/81



## **EQA for Mindray Testing System—— CAP**

Mindray testing system has passed CAP for 6 continuous years

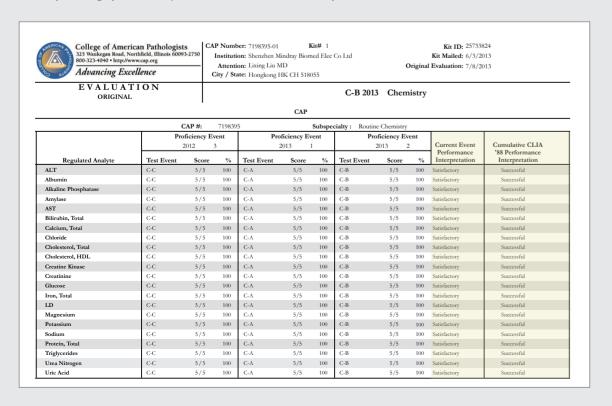

#### Reagent menu

#### **Hepatic Panel**

Alanine Aminotransferase (ALT)

Aspartate Aminotransferase (AST)

Alkaline Phosphatase (ALP)

y-GlutamylTransferase (y-GT)

Direct Bilirubin (D-Bil) DSA Method

Direct Bilirubin (D-Bil)VOX Method

Total Bilirubin (T-Bil) DSA Method

Total Bilirubin (T-Bil)VOX Method

Total Protein (TP)

Albumin (ALB)

Total Bile Acids (TBA)

Prealbumin (PA)

Cholinesterase (CHE)

α-L-fucosidase (AFU)

5'-nucleotidase (5'-NT)

#### **Renal Panel**

Urea (UREA)

Creatinine (CREA) Modified Jaffé Method

Creatinine (CREA)Sarcosine Oxidase Method

Uric Acid (UA)

Carbon dioxide (CO2)

Microalbumin

β2-Microglobulin (β2-MG)

Cystatin C (CysC)

Retinol binding protein(RBP)

#### **Immune Panel**

Immunoglobulin A (IgA)

Immunoglobulin G (IgG)

Immunoglobulin M (IgM)

Immunoglobulin E (IgE)

Complement C3 (C3)

Complement C4 (C4)

#### **Diabetes Panel**

Glucose (Glu) GOD-POD Method

Glucose (Glu) HK Method

Hemoglobin A1c (HbA1c)

Fructosamine (FUN)

β-Hydroxybutyrate(β-HB)

#### Cardiac panel

Creatine Kinase (CK)

Creatine Kinase-MB (CK-MB)

Lactate Dehydrogenase (LDH)

 $\alpha$ -Hydroxybutyrate Dehydrogenase( $\alpha$ -HBDH)

High sensitive C-reaction protein( HS-CRP)

Homocysteine (HCY)

Myoglobin(MYO)

D-Dimer(D-Dimer)

#### Inorganic & Anemia

Iron (Fe)

Ferritin (FER)

Transferrin (TRF)

Calcium (Ca)

Magnesium (Mg)

Phosphate Inorganic (P)

Unsaturated iron binding capacity (UIBC)

Glucose-6-phosphate dehydrogenase (G6PD)

#### **Lipid Panel**

Total Cholesterol (TC)

Triglycerides (TG)

HDL-Cholesterol (HDL-C)

LDL-Cholesterol (LDL-C)

Apolipoprotein A1 (ApoA1)

Apolipoprotein B (ApoB)

Lipoprotein(a) [Lp(a)]

#### **Rheumatism Panel**

C-reactive protein (CRP)

Rheumatoid Factor (RF)

Antibodies Against Streptolysin O (ASO)

#### **Lung Panel**

Adenosine Deaminase (ADA)

Angiotensin Converting Enzyme(ACE)

#### **Pancreatitis Panel**

α-Amylase (α-AMY)

Lipase (LIP)